

# UROLIFT®

BPH Relief. In Sight.™



*Not actual patient*

## Patient Information

The UroLift® System is a proven approach to treating enlarged prostate without heating, cutting or removing tissue<sup>1-7</sup>


# What is an **enlarged prostate**?

Benign Prostatic Hyperplasia, or BPH, is a common condition in which the prostate enlarges as men get older. Over 40% of men in their 50s and over 70% of men in their 60s have BPH.<sup>1</sup> While BPH is a benign condition and unrelated to prostate cancer, it can greatly affect a man's quality of life.


**As the prostate enlarges, it presses on and blocks the urethra, causing bothersome urinary symptoms such as:**

- Frequent need to urinate both day and night
- Weak or slow urinary stream
- A sense that you cannot completely empty your bladder
- Difficulty or delay in starting urination
- Urgent feeling of needing to urinate
- A urinary stream that stops and starts

**Normal Prostate**

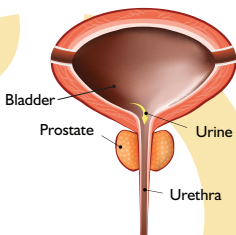

**Benign Prostatic Hyperplasia (BPH)**

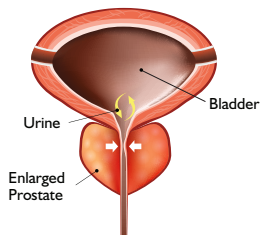



*Not actual patient*




# What is the UroLift® System?

The UroLift® System is a minimally invasive procedure to treat an enlarged prostate. It is an option for patients looking for an alternative to medications or major surgery. It does not require heating, cutting or destruction of prostate tissue.<sup>1-7</sup>

## Advantages of the UroLift® System

- ✓ Safe and effective<sup>8</sup>
- ✓ Risk profile better than reported for procedures such as TURP<sup>9</sup>
- ✓ Rapid symptom relief,<sup>10</sup> better than reported for medications<sup>11</sup>
- ✓ A BPH procedure shown to not cause new and lasting sexual dysfunction\*<sup>11-14</sup>

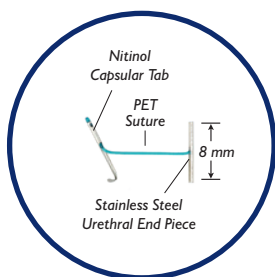

**ACTUAL SIZE**  
of UroLift® implant

\*No instances of new, sustained erectile or ejaculatory dysfunction in the L.I.F.T.<sup>15</sup> pivotal study

1. Lokeshwar, Transl Androl Urol 2019; 2. Miano, Med Sci Monit 2008;  
3. Mirakhur, Can Assoc Rad J 2017; 4. Woo, BJUI 2011; 5. McVary, J Urol 2015;  
6. Gilling, Can J Urol 2020; 7. Kadner, World J Urol 2020; 8. Roehrborn Urology Clinics 2016;  
9. Sonksen, EU Urol, 2015; 10. Roehrborn. J Urology 2013;  
11. AUA BPH Guidelines 2003, 2010, 2018, amended 2019;  
12. Naspro, Eur Urol 2009; 13. Montorsi, J Urol 2008; 14. McVary, J Sex Med 2014  
15. Roehrborn et al. Can J Urol 2017 5-Year LIFT Study.

Warning: This device contains stainless steel and nitinol, an alloy of nickel and titanium. Persons with allergic reactions to these metals may suffer an allergic reaction to this implant. Prior to implantation, discuss any allergy/hypersensitivity to these materials with your physician.

# How the UroLift® System works



## Step 1

The UroLift Delivery Device is placed through the obstructed urethra to access the enlarged prostate.



## Step 2

Small UroLift implants are permanently placed to lift and hold the enlarged prostate tissue out of the way and increase the opening of the urethra. The permanent Implants are delivered through a small needle that comes out of the UroLift Delivery Device and into the prostate.



## Step 3

The UroLift System treatment provides quick symptom relief by opening the obstructed urethra.



## What to expect during the procedure

- If the UroLift® System is right for you, your doctor will provide you with detailed information relating to the procedure.
- The doctor introduces a thin tube through the urethra and places implants into the prostate until a sufficient opening for urine to pass easily is achieved.



*Urologist's view of  
obstructed prostate*

---

## What to expect after the procedure

Most common side effects are temporary and include painful urination, blood in the urine, pelvic pain, urinary urgency and/or the inability to control the urge. Rare side effects, including bleeding and infection, may lead to serious outcomes and may require intervention. Most symptoms typically resolve within two to four weeks after the procedure.<sup>1</sup>



*Unblocked urethra after  
UroLift System procedure\**

\* Results may vary  
1. Roehrborn, J Urology 2013 LIFT Study



*Not actual patient*

# FAQ'S

## PROCEDURE

### **How do I determine if I may be a candidate for the UroLift® System procedure?**

*Speak to your doctor to determine if you are a candidate for the procedure. You may be a candidate if you are a male, 50 years of age or older, and have symptoms relating to BPH.*

### **How long will my results last?**

*The UroLift System procedure is intended to be a long-lasting solution for BPH/LUTS that utilizes a permanently placed implant. Clinical studies have proven durability out to five years.<sup>1</sup> Individual results may vary.*

### **Will my prostate continue to grow even after I get a UroLift System procedure?**

### **What if I need another procedure?**

*Your prostate may continue to grow after the procedure; however, receiving the UroLift System treatment does not preclude subsequent treatment with a UroLift System or other procedure.*



1. Roehrborn et al. Can J Urol 2017 5-Year LIFT Study;  
2. Roehrborn, J Urology 2013 LIFT Study





## WHAT TO EXPECT?

### **How quickly can I expect to have symptom relief?**

*Patients can experience minimal downtime post-treatment and symptom relief as early as two weeks.<sup>2</sup>*

### **When can I resume my usual activities?**

*Many men experience rapid recovery in days following the procedure.<sup>2</sup> Your doctor will discuss any restrictions and your specific situation after your procedure.*

### **Will I need to continue taking my BPH medications after the procedure?**

*The goal of the UroLift System procedure is to provide the relief you need to discontinue your BPH medications. Your doctor will decide if continued use of BPH medication is necessary.*

### **Will my sexual function be affected by the UroLift® System procedure?**

*Clinical studies have shown the UroLift System procedure does not cause instances of new, sustained erectile or ejaculatory dysfunction.<sup>\*2</sup> The same cannot always be said of other BPH therapies such as TURP, laser, and even medication.<sup>3</sup>*

### **Can I have an MRI after the UroLift System procedure?**

*The UroLift implant is MR conditional and patients can be safely scanned after a UroLift System treatment. Ask your doctor about these MR conditions.*

\*No instances of new, sustained erectile or ejaculatory dysfunction in the L.I.F.T. pivotal study  
2. Roehrborn, J Urology 2013 LIFT Study; 3. AUA BPH Guidelines 2003



## Now what?

Are you ready to take control of your BPH symptoms? Don't settle for frustrating urinary symptoms and daily medications. The UroLift® System procedure may be the right choice for you if you are seeking a treatment solution. So now what?

1. **Talk to your doctor** to see if you are a candidate for the UroLift System procedure
2. **Your doctor** may recommend a series of diagnostic tests to evaluate your condition

**[teleflex.link/Urolift.BR](https://teleflex.link/Urolift.BR)**



Not actual patient

# UROLIFT®

**Visit [teleflex.link/Urolift.BR](https://teleflex.link/Urolift.BR)**  
**Customer Service: [la.cs@teleflex.com](mailto:la.cs@teleflex.com)**

Follow us on  **@TeleflexLATAM**

Find us on  **[facebook.com/TeleflexLATAM](https://facebook.com/TeleflexLATAM)**

NeoTract, Inc. is dedicated to developing innovative, minimally invasive and clinically effective devices that address unmet needs in the field of urology. Our focus is to improve the standard of care for patients with Benign Prostatic Hyperplasia (BPH), a broadly underserved market.

As with any medical procedure, individual results may vary. Speak with your urologist to see if the UroLift® System procedure is right for you. Most common side effects are temporary and include painful urination, blood in the urine, pelvic pain, urinary urgency and/or the inability to control the urge. Rare side effects, including bleeding and infection, may lead to serious outcomes and may require intervention.

The products in this document may not be available in all countries. Please contact your local representative. Please check your local regulatory approval status. Refer to the applicable Instructions for Use for the indications approved in your geography.

BPH Relief. In Sight. UroLift, the UroLift logo, NeoTract, the NeoTract logo, Teleflex and the Teleflex logo are trademarks or registered trademarks of Teleflex Incorporated or its affiliates, in the U.S. and/or other countries. Registro ANVISA: 80117580890 and 80117589018 © 2021 NeoTract, Inc. All rights reserved. MC-007138 LA EN  
Contact: [la.cs@teleflex.com](mailto:la.cs@teleflex.com)

**neo**tract**®** | **Teleflex®**  
INTERVENTIONAL UROLOGY

Teleflex · 3015 Carrington Mill Boulevard, Morrisville, NC 27560